

ChicagoTip™

Self Cleaning Surgical Suction Tip

DESIGN

- Ergonomic Design
- Stainless Steel Tip
- Disposable Self Cleaning Suction Tip
- Quiet

INSTRUCTIONS FOR USE

Step 1: Push Slider forward to remove any obstruction causing the clog.

Step 2: Push slider back in place to resume suction.


STERILIZATION GUIDELINE

Sterilized by radiation.

CHICAGOTIP™ ADVANTAGES

- Allows the surgeon to unclog the tip by pushing a retractable stylet in the handle.
- Has a steel tip resistant to chipping of the tip from a high speed drill.
- Has a tear drop vent system in the handle rendering the suction very quiet.
- Single use- reduces the chance of infection.

- Sizes: 10 and 12 French
- #NE201 - 10 French
- #NE202 - 12 French
- Made in USA
- "Patent" www.neuroenterprises.com/patent

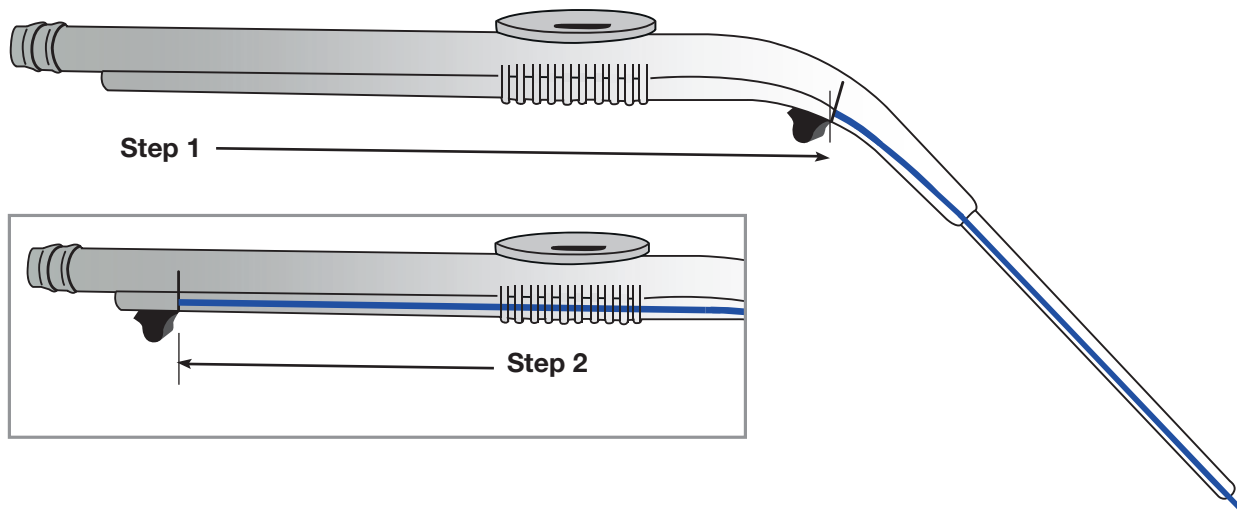

**NEURO
ENTERPRISES**

DESIGNS BY SURGEONS FOR SURGEONS

Manufactured for NEUROENTERPRISES
15 Riverwalk Ct, Port Barrington, IL 60010 • Tel: 312.859.6889 • Fax: 866.231.5454

FOR FURTHER INFORMATION CALL

312.859.6889

TO PLACE ORDERS FAX OR EMAIL

866.231.5454

KHAMBURG@NEUROENTERPRISES.COM

(Carton Contains 10 Suction Tips)
Minimum Order 1-Carton